

DREAMLAND


REVIVE ME
HEAT PAD

TYPE R6705
R6707
R6717



Instructions for use

ILLUSTRATIVE GUIDE

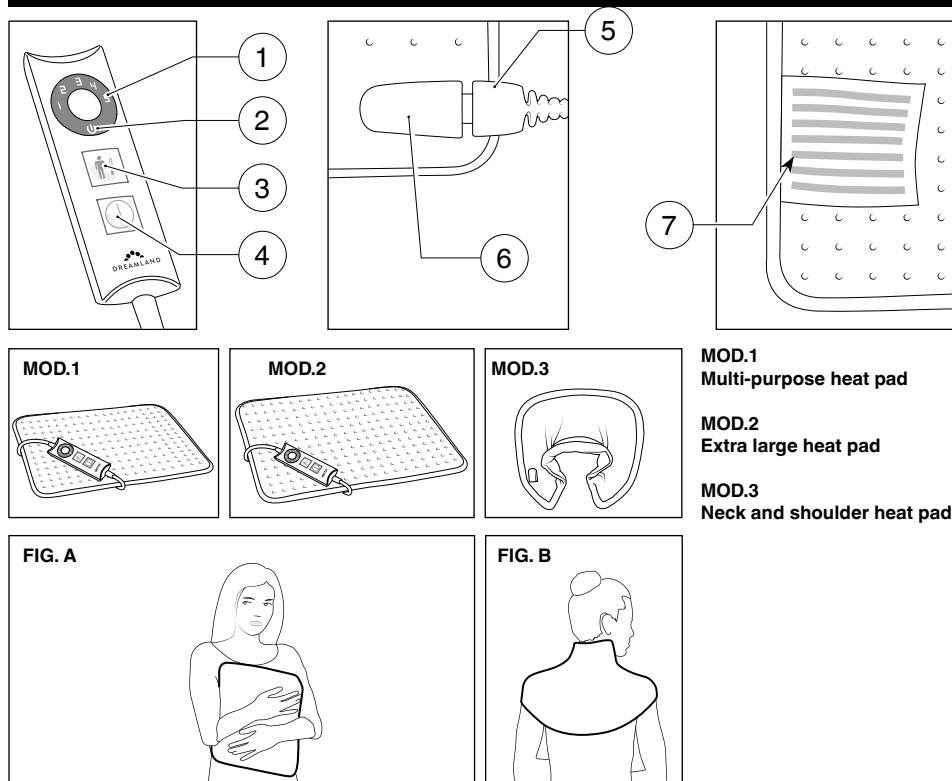

DESCRIPTION OF THE APPLIANCE

- | | |
|-------------------------|---------------------------|
| 1. Temperature leds | 5. Control unit connector |
| 2. Stand-by led | 6. Heating pad connector |
| 3. Temperature selector | 7. Product label |
| 4. ON/OFF selector | |

TECHNICAL DATA

TYPE R6705
220-240 V
50/60 Hz 110 W

TYPE R6717
220-240 V
50/60 Hz 110 W

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











TYPE R6707
220-240 V
50/60 Hz 110 W

Dear Customer, thank you for choosing our product. We feel certain you will appreciate its quality and reliability as it was designed and manufactured with customer satisfaction in mind. These instructions for use are compliant with the European Standard EN 82079.

INDEX

Illustrative guide	page 1
Description of the appliance	page 1
Technical data	page 1
Symbols	page 2
Important instructions	page 3
Use	page 6
Cleaning and maintenance	page 7
Storing the heating pad	page 8
Troubleshooting	page 8
Disposal	page 9
Guarantee	page 9

SYMBOLS


 <i>Protection class II</i>	 <i>Prohibited</i>
 <i>Carefully read the instruction manual</i>	 <i>Important note</i>
 <i>Warning</i>	 <i>Not suitable for children under the age of 3</i>
 <i>Do not insert pins</i>	 <i>Do not bleach</i>
 <i>Wash at 30°C</i>	 <i>Do not dry-clean</i>
 <i>Dry in a tumble dryer on a cycle at a moderate temperature</i>	 <i>Do not iron</i>

IMPORTANT INSTRUCTIONS.

RETAIN FOR FUTURE USE.



WARNING! Instructions and warnings for safe use.

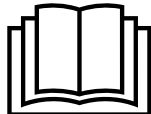

Before using this appliance, carefully read the instructions for use and in particular the safety notes and warnings, which must be complied with. Keep this manual together with its illustrative guide for future consultation. Should you transfer the appliance to another user, make sure you hand over this documentation as well.

ⓘ NOTE: should any part of this manual be difficult to understand or any doubt arise, contact the company at the address indicated on the last page before using the appliance.

ⓘ NOTE: Persons with impaired vision may consult these instructions for use in digital format at the website www.dreamlanduk.co.uk

- After unpacking, make sure the appliance is intact, complete with all its parts as shown in the illustrative guide and with no sign of damage. If in any doubt, do not use the appliance and contact an authorised service centre.



WARNING! Risk of suffocation.

Children shall not play with the packaging. Keep the plastic bag out of the reach of children.

- This appliance must be used only for the purpose for which it was designed, namely as a pad for home use. Any other use is considered not compliant with the intended use and therefore dangerous.
- Before connecting the appliance to the mains supply, make sure that the voltage and frequency indicated on the technical

data of the appliance correspond to those of the mains supply. The rating is on the appliance or on the power supply unit, if any (see illustrative guide and technical data).

- This appliance can work automatically at a frequency of 50 Hz or 60 Hz.
- This appliance is only to be used with the types of detachable control unit that are marked on the appliance.



Children under the age of three are not to use this appliance due to their inability to react to overheating.

- This appliance can be used by children older than 3 years and younger than 8 years under supervision and with the control always set to minimum temperature (heat setting 1).
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.
- This appliance must not be used by persons insensitive to heat and other very vulnerable persons who are unable to react to overheating.
- The cable and control unit of the appliance can give rise to risks of entanglement, strangulation, tripping or treading if not correctly arranged. The user shall make sure that electric cords shall be arranged in a safe way.
- Always disconnect the appliance from the mains supply before cleaning or performing maintenance and when the appliance is not in use.
- Examine the appliance frequently for signs of wear or damage. If there are such signs, if the appliance has been misused or does not work, contact an authorised technical service centre before switching it on again.
- If the supply cord is damaged, take the whole appliance to an

authorised technical service centre for the replacement of the control unit, so as to prevent all risks.

- This appliance is not intended for medical use in hospitals.
-  DO NOT use if wet.



DO NOT INSERT PINS OR NEEDLES.



 When operating the appliance over a prolonged period of time **DO NOT** exceed the temperature levels stated for continuous usage.

If the appliance is set and used at a temperature higher than those indicated for continuous usage over an extended period of time, the user may suffer from skin burns or heat-related injuries.

- When storing the appliance, allow it to cool down before folding.







WARNING! DO NOT crease the appliance by placing items on top of it during storage.



WARNING! Follow indications in the relevant chapters of this manual on how to use, clean, maintain or store the appliance.



WARNING! DO NOT let the heating pad come into direct contact with the skin.

- The use of creams or ointments combined with the heat of the appliance may increase the risk of skin burns.
- Persons with a pacemaker must **NOT** use this appliance.
-  **DO NOT** use the appliance with wet hands, damp feet or barefoot.
-  **DO NOT** pull the supply cord or the appliance itself to remove the plug from the socket.
-  **DO NOT** pull or lift the appliance by the supply cord.
-  **DO NOT** expose the appliance to humidity, to the effects of weather (rain, sun, etc.) or to extreme temperatures.
- If the appliance is faulty or malfunctions, switch it off and do

not tamper with it. Contact an authorised technical service centre for any repairs.

- This appliance shall not come into contact with water, other liquids, sprays, steam. For any cleaning and maintenance operation refer to the instructions in the relevant chapter.
- Ⓞ **DO NOT use this appliance to warm animals.**
- See the external packaging for the characteristics of the appliance.

USE

- Insert the control unit connector (5) into the heating pad connector (6) to connect the control to the heating pad.
- Insert the plug in the mains socket.
- Place the heating pad (MOD.1 or MOD.2) on the part of the body that is to be heated (FIG. A). Place the neck and shoulder heat pad (MOD.3) around your neck and secure in place with the velcro fastening (FIG.B).

SWITCHING ON AND SELECTING THE RIGHT TEMPERATURE

The control unit can be set to the following temperature levels:

5: Fast pre-heat temperature

4-3-2: Medium temperatures

1: Temperature for prolonged use

- Switch the heating pad on by pressing the ON/OFF button (4). The temperature is automatically set at level 5, so LED 5 (1) will light up on the control unit.
- To heat the heating pad quickly, use the maximum temperature 5. You can then set a lower temperature level, depending on the degree of heat required. To change the temperature, press the temperature selector (3) until the LED (1) corresponding to the required level lights up on the control unit. The higher the number selected, the higher the temperature.
- If the heatpad is used for a prolonged time, or when it's likely that you might fall asleep with the heatpad still switched on, use temperature level 1.

AUTOMATIC SHUTDOWN

- The heating pad will provide a 3 hour heat treatment, then it will automatically shut off. The standby led (2) will light up after the 3 hours of operating to indicate that the product has switched off automatically.

- To switch the appliance back on by pressing the ON/OFF selector (4) twice and select the desired temperature once again.

SWITCH-OFF

- Switch the heating pad off by pressing the ON/OFF selector (4) when it is no longer used.
- Remove the plug from the mains socket.

CLEANING AND MAINTENANCE

The heating pad can be washed. Refer to the washing symbols on the product and cover labels for the washing instructions.



WARNING! Always disconnect the appliance from the mains and disconnect the control connector from the appliance before cleaning or performing maintenance.

BEFORE WASHING

- Disconnect the power cable from the socket.
- Disconnect the heating pad connector (6) from the control unit connector (5).



MACHINE WASHING

- The heating pad can be machine washed at 30°C with a normal cycle.
- Use the gentle spin cycle only. DO NOT run an additional spin cycle.



TUMBLE DRYING

- The heating pad can be dried in a tumble dryer on a delicate cycle, but at the end of the drying cycle it must be air-dried to allow it to dry completely.
- Remove the heating pad from the tumble dryer at the end of the drying cycle and reshape it to its original size.
- Allow the heatpad to fully air-dry before using it again.

AIR DRYING

- Lay the heating pad flat on a clothes airer until completely dry.



WARNING! Dry the appliance flat without using pegs to avoid damaging it. Leave it to dry, laid out flat on a clothes airer, not in direct sunlight.

-  DO NOT iron the heating pad.

Store this appliance as follows when not used:

- Wait for the heating pad to be completely dry if previously washed.
- It is recommended to place the heating pad in the package provided upon purchase for better storage.
- When storing the appliance, allow it to cool down before folding.
- Store the heating pad in a dry place.

Examine the appliance and its supply cord frequently for signs of wear or damage. If there are such signs or in the case of malfunction, contact an authorised technical service centre.

TROUBLESHOOTING

This chapter describes the most common problems that can arise using the appliance. If the problems cannot be solved with the information below, please contact the Authorised Service Centre.

Problem	Solution
The LEDs on the control unit do not light up.	Check that the control unit plug is fully inserted in the socket. If the LEDs remain off, remove the control unit plug from the socket and return the underblanket to an authorised technical service centre.
The blue warning light is flashing.	Turn the control unit off and remove the plug from the socket. Check that the control unit connector (5) is fully inserted in the heating pad connector (6) and reinsert the plug in the socket. If the standby LED continues to flash, return the heating pad to an authorised technical service centre.
Temperature LED 1 (1) is flashing.	Return the heating pad to an authorised technical service centre to be checked.
The product has been used incorrectly or is damaged.	
The power cable or another internal wire is damaged.	
The control unit is damaged.	
There are holes or tears in the fabric or stubborn creases that do not spread out.	

DISPOSAL



The packaging of the appliance is made of recyclable materials. Dispose of the packaging in accordance with the environmental protection regulations.



Pursuant to Directive 2012/19/EU on waste electrical and electronic equipment (WEEE), the crossed bin symbol on the appliance or its packaging indicates that the appliance must be disposed of separately from other waste at the end of its useful life.

The user must therefore take the appliance to an appropriate recycling centre for electrical and electronic equipment. Alternatively, instead of handling the disposal, the user can hand over the appliance to the retailer when a new, equivalent one is purchased. The user can hand over electronic products measuring less than 25 cm to retailers of electronic products having a sales area of at least 400 m² free of charge and with no obligation to buy. Proper waste sorting prior to sending the equipment no longer used for recycling, processing and disposal in a manner that respects the environment contributes towards avoiding negative impacts on the environment and on health and promotes the reuse and/or recycling of the equipment's material.

GUARANTEE

This product is guaranteed for a period of 3 years from the date of purchase against material, mechanical and electrical defects. The guarantee is binding if it is accompanied by a valid receipt of payment. This guarantee is only valid if the appliance is used solely for domestic purposes in accordance with the instructions provided that is not connected to an unsuitable electricity supply, dismantled, interfered with in any way damaged, through misuse, fair wear and tear. Under this guarantee we undertake to repair or replace free of charge any parts found to be defective.

PLEASE NOTE, THE COST OF A SAFETY INSPECTION IS NOT COVERED BY THIS GUARANTEE. A CHARGE WILL BE MADE IN RESPECT OF ANY PRODUCT RETURNED UNDER THIS GUARANTEE, WHERE NO FAULT IS FOUND.

This guarantee is offered as an extra benefit and does not affect your statutory rights as a consumer. This guarantee does not confer any rights other than those expressly set out above and in particular we shall not be liable for any indirect or consequential loss or damage. Nothing in this guarantee or the instructions relating to the product excludes, restricts or otherwise affects your statutory rights. In line with our policy of continuous development, we reserve the right to change this product, packaging and documentation without notice. If a defect appears and after checking that the blanket is being used properly in accordance with the instructions, return it with a valid proof of purchase, as soon as possible to the manufacturer's Service Centre. This guarantee is valid in the UK only.

SERVICE CENTRE

PO Box 166 – Middlewich – Cheshire – CW10 0ZU - Tel: 0333 0050305

E-mail: customerservice@imetec.co.uk

www.dreamlanduk.co.uk

MID04052
161221

